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HI

AR

**CLASS
MEDICAL**
Simplicity that works...

CE
0050

Class Medical Ltd
Transurethral Catheter Safety Valve (TUC Safety Valve),
Adult Type
Sterile, for single use.

PLEASE READ ALL INSTRUCTIONS CAREFULLY BEFORE USE!

 **Warning:**

“Adult type” Transurethral Catheter Safety Valves are not intended for patients aged under 16 years.

Product description

All Transurethral Catheter Safety Valves (TUC Safety Valves) basically comprise of a proximal female Luer lock, a flow restrictor, pressure release valve and a distal male Luer slip.

Indications

The TUC Safety Valve is indicated for single use during inflation of a Foley catheter retaining balloon, to prevent accidental inflation of the balloon in the urethra.

Contraindications

The Adult type TUC Safety Valve is not intended for patients aged under 16 years of age.

The TUC Safety Valve is not intended for use as part of the removal of the catheter procedure.

Preparations for use

Remove TUC Safety Valve from its sterile packaging using aseptic technique.

Visually inspect the TUC Safety Valve for any mechanical damage.

Remove red pull tab prior to inflation of catheter retaining balloon.

Instructions for Use

Before inserting the catheter, using sterile technique, connect the syringe to the balloon inflation port of the catheter and test inflate the balloon with 2-3ml of fluid to evaluate balloon integrity. Upon successful evaluation, empty the balloon and connect the female luer lock/slip of the TUC Safety Valve to the syringe and the male luer slip of the TUC Safety Valve to the balloon inflation port. Continuing the use of a sterile technique, insert Foley catheter as per standard procedure/instructions.

Remember that balloon inflation with the TUC Safety Valve in place is slower (due to the restriction channel) and will take 10-15 seconds longer than usual.

Upon completion of inflation **immediately disconnect the TUC Safety Valve and syringe assembly together** from the balloon inflation port.

 **Warning:**

 **“STOP, IF YOU SEE A DROP!”**

If fluid leaks from the exhaust port of the valve, the catheter anchoring balloon is not in the bladder. In this event, stop depressing the syringe, aspirate the balloon fluid back into the syringe, reposition the catheter, ensuring the retaining balloon is placed in the bladder, and attempt inflation again. If the valve exhausts again call for expert urological assistance.

 **Warning:**

In the event of valve activation, it is important to note how much fluid remains in the syringe to ensure that the balloon is ultimately filled with a volume in accordance with the catheter manufacturers guidelines.

 **Warning:**

Ensure that the Luer-lock connection between the syringe and the TUC Safety Valve and the push-in connection between the TUC Safety Valve and the balloon inflation port are sufficiently tight to prevent leaks and/or false positives, as per a normal catheterisation procedure.

 **Warning:**

Failure to remove the TUC Safety Valve from the catheter may cause deflation of the retaining balloon and subsequent catheter balloon migration into the urethra or complete loss of the catheter.

 **Warning**

The TUC Safety Valve is intended only for urological use.

Never block the safety valve exhaust ports. There is a risk of inflation of the retention balloon in the urethra causing urethral trauma.

Note:

The TUC Safety Valve is not to be used for the removal or deflation of transurethral catheters.

Potential Complications

The restriction orifice of the TUC Safety Valve may become blocked preventing the depression of the syringe plunger.

If difficulty is encountered depressing the syringe, remove the TUC Safety Valve from the catheter and test flow through it. If no fluid passes through the TUC Safety Valve the restrictor orifice is blocked. Dispose of the TUC Safety Valve.

Caution

DO NOT REUSE

Re-processing of medical devices intended for single use only may result in degraded performance or a loss of functionality. Re-use of single use only medical devices may expose patient or user to viral, bacterial, fungal, or prionic pathogens. Validated cleaning and sterilisation methods and instructions for reprocessing to original specifications are not available for these medical devices. This product is not designed to be cleaned, disinfected, or re-sterilised.

Delivery

TUC Safety Valves are supplied individually packaged in a sterile pouch.

The sterility of these products is only guaranteed as long as the sterile pouch is unopened and undamaged, and only until the expiry date indicated.

Storage Instruction

The products should be stored in their original packaging in a dry place, preferably away from direct and indirect sources of light and heat.

Keep away from sunlight and keep dry. Do not use if the sterile pouch is compromised.

Disposal

After use, the TUC Safety Valve may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

Incident Reporting

Adverse Events associated with the TUC Safety Valve should be reported to the Manufacturer or Distributor without undue delay. Serious Adverse Events (such as the possibility of death or serious injury) should also be reported **immediately** to the Manufacturer or Distributor and relevant Competent Authority.

Explanation of important symbols and markings on the product label.

Qty	Quantity, Number of items
	Do not use if package is damaged
	For use with patients aged 16 and over

**Class Medical Ltd., Unit 1D, Annacotty Business Park,
Annacotty, Co. Limerick. IRELAND**

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